

148.50 mm

KN95 FACE MASK INSTRUCTION FOR USE

TYPE:SF/WKZ-2P

How To Use:



Step 1: The nose clip part is facing up, the side without nose clip is facing the face, and the ear straps are pulled on each hand.

Step 2: Face the mask against the chin first, and pull the ear band back to the ear and hang it on the ear until it feels comfortable.

Step 3: Place the fingers of both hands in the middle of the nose clip and press inwardly according to the shape of the nose bridge to conform to the shape of the nose bridge.

Step 4: Put your hands on your face when using, and check the tightness with your face.

Change the mask if there is damage, contamination or difficulty in breathing.

Warning

If instructions are not followed as recommended it may lead to poor performance of mask and may cause health risks.

- IP non-woven fabric, high-efficient electrostatic melt-blown fabric.
- 3D shape is an ergonomic design according to human face to ensure the tightness and an easier breathing for healthier protection.
- Nose clip and flexible ear straps for a comfortable wearing.
- 95% filter efficiency against solid and liquid particulates, excluding oils.
- This respirator is designed for use for particles such as those from grinding, sanding, sweeping, sewing, logging, or other dusty operations. This respirator can also help reduce inhalation exposures to certain airborne biological particles.
- Non-allergic, latex-free.

Symbol description



single use only



batch number



expiry date



Manufacturing date

210.00 mm